



## INSTRUCTIONS FOR USE

### GINGI SEP

**Product:** GINGI SEP

**Description:** Separator for silicone surfaces - gingiva mask

**Usage:** Optimal to avoid adhesion between different types of silicones

**Instructions:**

- Make sure that the surfaces are clean and dry.
- Use the brush to spread the separator.
- Wait for the separator to dry completely, until the surface becomes opaque.
- Use a gentle blow of air to reduce the drying time.
- Close the bottle tightly after use.

**Guarantee:** Our recommendations for use are based on our tests and experience. Due to the fact that that our products are in a continuous development and improvement, variations in composition is reserved.

**Colour:** clear

**Packaging:** 15 ml bottle

**Terms of validity:** 2 years