



INSTRUCTIONS FOR USE

THERMO SEP

Product: THERMO-SEP

Description: Separating liquid plaster-thermoplastic resin

Usage: Optimal for the separation plaster-thermoplastic resin

Instructions:

- Wash the flask.
- Wait until the flask has an optimum temperature (warm).
- Apply a thin layer with the brush and pay attention to the spaces between teeth.
- Wait 15 minutes and then you can put it in the injection machine.

Guarantee: Our recommendations for use are based on our tests and experience. Due to the fact that that our products are in a continuous development and improvement, variations in composition is reserved.

Colour: light orange

Packaging: 500 ml bottle

Terms of validity: 2 years